

QUICK START GUIDE

Cervico Essential Mold

Benchtop mold for chairside customization of stock temporary abutments into anatomical healing abutments, using the implant system's stock lab analogs.

REF 5213006651779

VP INNOVATO HOLDINGS LTD
Poseidonos 4A, Aradippou 7101, Larnaca, Cyprus
<https://innovatoholdings.com> · +357 99 883 898



In the box


- Aluminium mold ring with well codes
- Silicone insert: 24 anatomical wells

6 steps

- 01


Clean and sterilize first
Non-sterile on delivery. Separate silicone from base, ultrasonic clean 10 min at 50 °C, rinse, dry, pouch, autoclave 4 min at 134 °C, dry 30 min.
- 02

Read the code
Shape a / p / m1 / m2, size S / M / L, c centered or o off-centered. m1Lc = lower molar, large, centered. The code comes from the Cervico Guide tab chosen in the mouth.

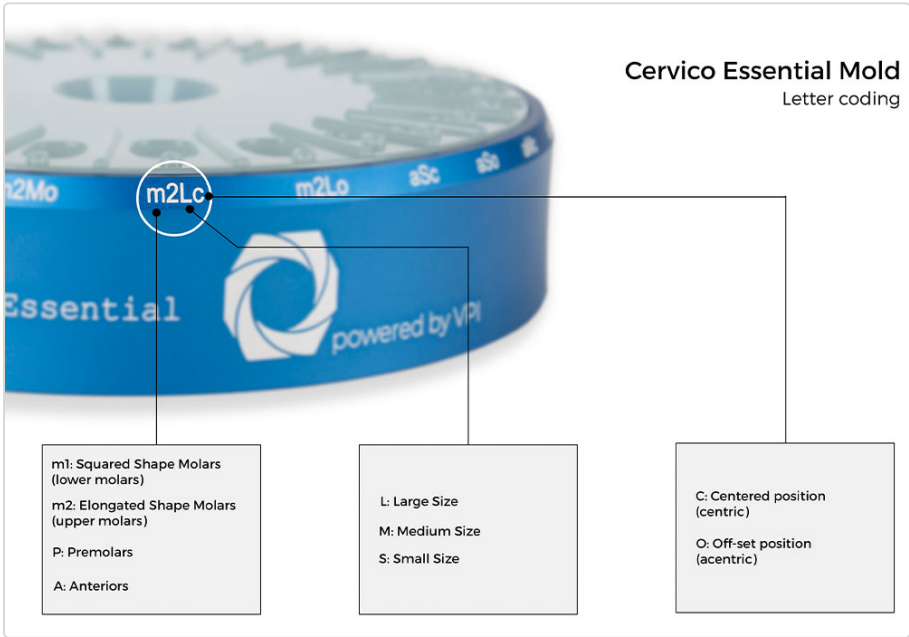


- 03

Abutment on analog, into the well
Temporary abutment on the system's lab analog. Sink the analog into the bottom hole until the abutment shoulder is level with the well floor.

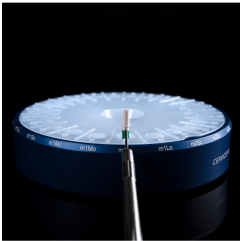


04 Turn to the tear

Rotate until the hex flat (or your buccal mark) faces the tear-shaped bump on the front of the well.

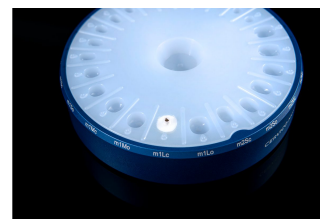

05 Fill and cure

Flowable nano-hybrid composite; large wells in two increments. 60 s in the well, 60 s out.

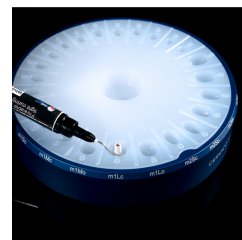

06 Out, polish, disinfect

Twist out with tweezers or back the screw out while pulling. Polish smooth. Remove the analog. Disinfect like any abutment.



Never skip

- ✗ The mold never goes in the mouth; the abutment does, so polish and disinfect it.
- ✗ Wrong rotation = wrong profile in the mouth. Always align to the tear.
- ✗ Max 145 °C. Replace a torn silicone insert.

After every patient

- Wipe wells with alcohol, take the silicone out of the base, tabs and pins off the handle.
- Ultrasonic clean 10 min at 50 °C, aldehyde-, phenol- and chlorine-free detergent. Rinse with purified water. Dry.
- Inspect. Pouch. Autoclave: 4 min at 132–134 °C pre-vacuum or 30 min at 121 °C gravity. Dry 30 min.
- Never above 145 °C. Never dry heat, chemiclave, EtO or plasma.

Full instructions

This card does not replace the Instructions for Use. Read **VPI-IFU-EM** before first use:

<https://vpicervico.com/instructions-of-use/>

VP INNOVATO HOLDINGS LTD · Poseidonos 4A, Aradippou 7101, Larnaca, Cyprus · +357 99 883 898 ·
info@vpicervico.com

REF 5213006651779 · Class I medical device, CE · Rx only (USA) · Non-sterile, sterilize before use · VPI-QSG-EM rev 2.0,
2026-09-09