

QUICK START GUIDE

Cervico Guide

Reusable intraoral and extraoral evaluation instrument for selecting the cervical profile of a custom healing abutment and for centering the implant osteotomy.

REF 5213006650017

VP INNOVATO HOLDINGS LTD

Poseidonos 4A, Aradippou 7101, Larnaca, Cyprus

<https://innovatoholdings.com> · +357 99 883 898

In the box

- Silicone case
- 12 anatomical tabs (a, p, m1, m2 × S, M, L)
- 8 cylindrical tabs 5–12 mm
- 4 guide pins
- Magnetic retention handle

6 steps

01 Clean and sterilize first


Non-sterile on delivery. Ultrasonic clean 10 min at 50 °C, rinse, dry, pouch, autoclave 4 min at 134 °C, dry 30 min.

02 Tie floss through every part that goes in the mouth

Tabs and pins have a bore for it. No exceptions.

03 Pick the tooth group, start with M

Mount the medium tab, try it in the site, then S or L. Thick biotype: crown width. Thin: one size smaller.

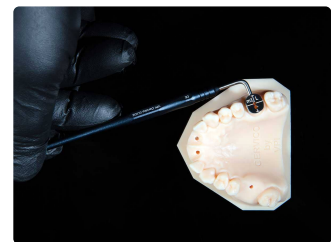

04 Chart the code and the T-line direction

E.g. "pS, hex flat facing buccal". The code is the mold well; the T-line is where the implant connection reference must face.

05 Center the osteotomy

Largest cylindrical tab that touches both neighbours, central line on the occlusal line, pilot drill below 2.1 mm through the bore.

06 Check every drill with a pin and the tab

Pin in the osteotomy, tab on the pin, centered or off-centered. Correct at the next drill. Place the implant with its connection facing the T-line.

Never skip

- ✗ Loose part in the mouth: refer immediately, chest X-ray.
- ✗ Max 145 °C. No dry heat, no chemiclave.
- ✗ Not for patients allergic to aluminium or stainless steel.

After every patient

- Wipe wells with alcohol, take the silicone out of the base, tabs and pins off the handle.
 - Ultrasonic clean 10 min at 50 °C, aldehyde-, phenol- and chlorine-free detergent. Rinse with purified water. Dry.
 - Inspect. Pouch. Autoclave: 4 min at 132–134 °C pre-vacuum or 30 min at 121 °C gravity. Dry 30 min.
 - Never above 145 °C. Never dry heat, chemiclave, EtO or plasma.
-

Full instructions

This card does not replace the Instructions for Use. Read **VPI-IFU-G** before first use:

<https://vpicervico.com/instructions-of-use/>

VP INNOVATO HOLDINGS LTD · Poseidonos 4A, Aradippou 7101, Larnaca, Cyprus · +357 99 883 898 ·
info@vpicervico.com

REF 5213006650017 · Class I medical device, CE · Rx only (USA) · Non-sterile, sterilize before use · VPI-QSG-G rev 2.0,
2026-09-09