

QUICK START GUIDE

Cervico Premium Kit

Cervico Guide plus Cervico Premium Mold with centered and off-centered silicone inserts and the VPI Prosthetic Connection Insert(s) ordered for your implant system.

REF 5213006650024

VP INNOVATO HOLDINGS LTD

Poseidonos 4A, Aradippou 7101, Larnaca, Cyprus

<https://innovatoholdings.com> · +357 99 883 898

In the box

CERVICO GUIDE

- Silicone case
- 12 anatomical tabs (a, p, m1, m2 × S, M, L)
- 8 cylindrical tabs 5–12 mm
- 4 guide pins
- Magnetic retention handle

CERVICO PREMIUM MOLD

- Rotating mold base (top ring with codes, bottom ring with 36 sockets)
- Centered silicone insert
- Screwdriver
- VPI Prosthetic Connection Insert(s) as ordered

Cervico Guide in 6 steps

01 Clean and sterilize first

Non-sterile on delivery. Ultrasonic clean 10 min at 50 °C, rinse, dry, pouch, autoclave 4 min at 134 °C, dry 30 min.

02 Tie floss through every part that goes in the mouth

Tabs and pins have a bore for it. No exceptions.

03 Pick the tooth group, start with M

Mount the medium tab, try it in the site, then S or L. Thick biotype: crown width. Thin: one size smaller.



04 Chart the code and the T-line direction

E.g. "pS, hex flat facing buccal". The code is the mold well; the T-line is where the implant connection reference must face.

05 Center the osteotomy

Largest cylindrical tab that touches both neighbours, central line on the occlusal line, pilot drill below 2.1 mm through the bore.

06 Check every drill with a pin and the tab

Pin in the osteotomy, tab on the pin, centered or off-centered. Correct at the next drill. Place the implant with its connection facing the T-line.

Cervico Premium Mold in 7 steps

01 Clean and sterilize first

Non-sterile on delivery. Remove silicone and inserts, ultrasonic clean 10 min at 50 °C, rinse, dry, pouch, autoclave 4 min at 134 °C, dry 30 min.

02 Fit your VPI insert once

Silicone out. Socket group by underline: none = crestal, 1/2/3 lines = 1/2/3 mm subcrestal. Arrow to the flat, screw from below. Write socket and insert on the info card. Silicone back in.




03 Turn the ring to the charted well

Well code from the Guide (e.g. pS) over your insert. Assistant task.

04 Abutment into the insert

Screw the temporary abutment into the VPI insert. Orientation and depth are now fixed.

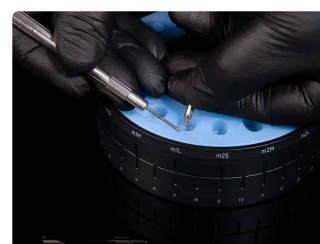


05 Fill and cure

Flowable nano-hybrid composite; large wells in two increments. 60 s in the well, 60 s out.




06 Out, polish, disinfect

Lift out. Polish smooth. Disinfect like any abutment before it goes in the mouth.

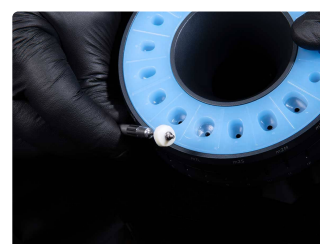

07 Impression post: same well, same insert

Stock impression post instead of the abutment gives the 1:1 duplicate for the impression.

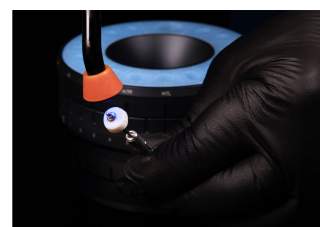

Never skip

- ✗ Loose part in the mouth: refer immediately, chest X-ray.
- ✗ Max 145 °C. No dry heat, no chemiclave.
- ✗ Not for patients allergic to aluminium or stainless steel.
- ✗ The mold never goes in the mouth; the abutment does, so polish and disinfect it.

- ✗ Wrong socket group = wrong depth. Check the underlines.
- ✗ Max 145 °C. Replace a torn silicone insert.

After every patient

- Wipe wells with alcohol, take the silicone out of the base, tabs and pins off the handle.
- Ultrasonic clean 10 min at 50 °C, aldehyde-, phenol- and chlorine-free detergent. Rinse with purified water. Dry.
- Inspect. Pouch. Autoclave: 4 min at 132–134 °C pre-vacuum or 30 min at 121 °C gravity. Dry 30 min.
- Never above 145 °C. Never dry heat, chemiclave, EtO or plasma.

Full instructions

This card does not replace the Instructions for Use. Read **VPI-IFU-PK** before first use:

<https://vpicervico.com/instructions-of-use/>

VP INNOVATO HOLDINGS LTD · Poseidonos 4A, Aradippou 7101, Larnaca, Cyprus · +357 99 883 898 ·
info@vpicervico.com

REF 5213006650024 · Class I medical device, CE · Rx only (USA) · Non-sterile, sterilize before use · VPI-QSG-PK rev 2.0,
2026-09-09